

FREE FORM BACK KITS

Unboxing . Assembly . Installation



INCLUDED IN THE BOX:

Free Form Shell, Air Mesh Cover, Mounting Hardware, Hex Keys, Fitting Manual, Extra Modules

PRE-FORM THE SHELL


Bend the Free Form shell to approximate the shape of the client's torso.



INSTALL THE CENTRE MOUNT

Loosen the two bolts on the clamp of the centre mount hardware. The silver plate slides onto the track from the bottom.

Tighten the two bolts to hold it in place on the track.

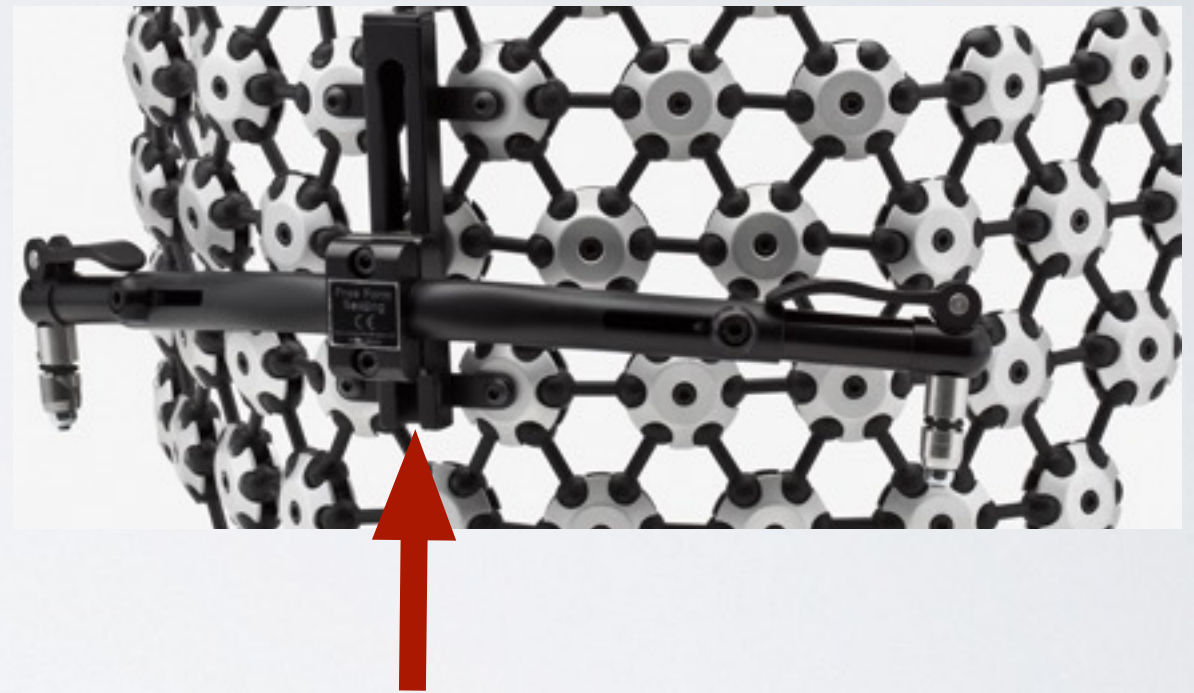

MOUNT RECEIVERS


Loosen the bolts to open the clamp and install onto the back canes. Set them at the same height.



MOUNT THE FREE FORM BACK TO THE CHAIR


Open the quick release lever and align the hardware. Once satisfied with the alignment, tighten the bolts. The lever tension can be adjusted with the nut on the bottom.



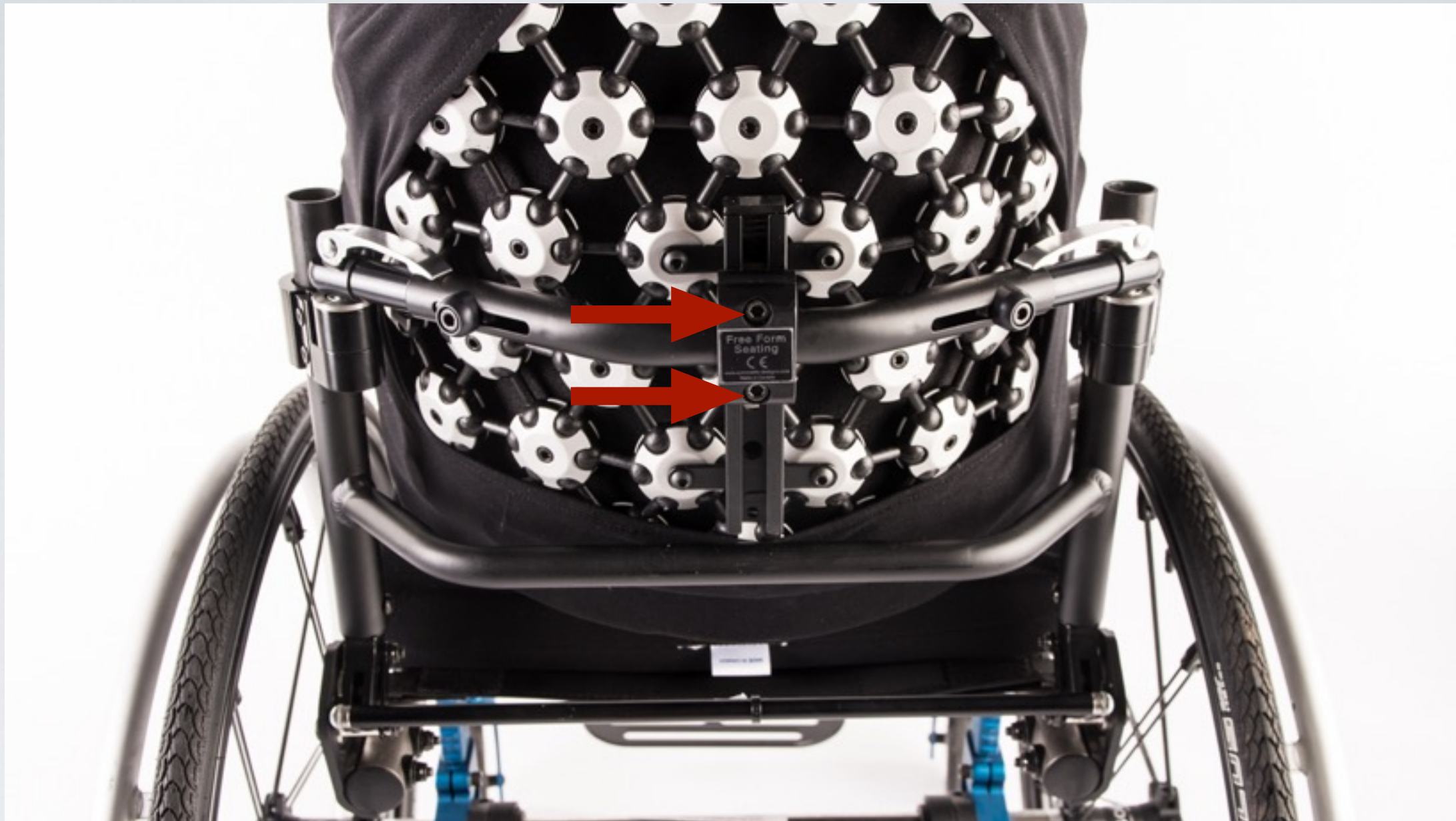

SET THE ANGLE, HEIGHT AND LATERAL POSITION OF THE SHELL

When loosened slightly, the two centre mount bolts allow for positioning of the shell in relation to the hardware

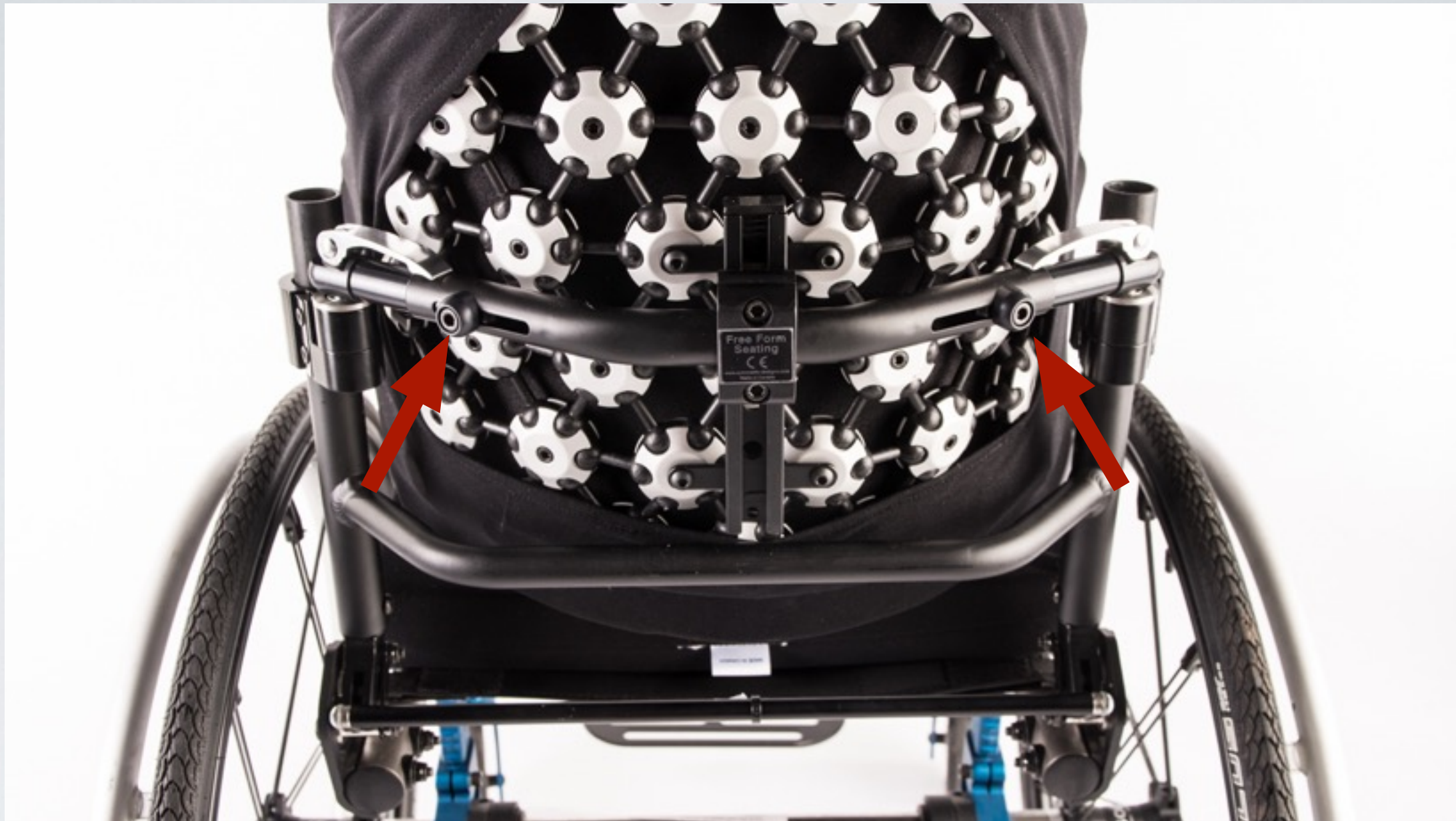

SET THE DEPTH OF THE SHELL

Loosen the horizontal bar telescoping bolts to change the width of the bar. When loosened, the mounting clamps can be rotated to change the depth of the shell.

READY FOR PATIENT TRANSFER

Inlay cover into the shell. It does not have to be mounted. Once the cover is placed in the shell, the patient can transfer into the chair.





TIGHTEN THE SHELL

Once satisfied with the contouring and shape, tighten each clamp using the 4mm hex key.